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Prüforganisation: TÜV Rheinland®

CP
Congratulations on your purchase of our exclusive bath accessory from the ATORE series.

Please note that the bath grip is for use at home and is only designed for supporting movements such as walking or supporting and standing up or sitting down.

Please read the installation instructions carefully and follow them exactly to ensure a secure and stable fixing of the ATORE bath grip.

Use the grip only as instructed. For it to achieve the specified limit load, it should either be form-fitted or friction-locked on walls, roofs or floors.

The fastening material provided is suitable only for use with the following types of walls:

Screws without dowel pins: for wooden walls

Screws with dowel pins: for concrete walls (full or porous concrete), stone walls (compound masonry, brick)

For surfaces other than the ones mentioned above, use suitable screws and dowel pins available in hardware stores. Ask an expert, keeping in mind the maximum load mentioned below.

Make certain that the base will be able to withstand the expected loads.

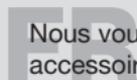
Do not use the grip anymore once it is damaged in any way!

Type: ATORE - grab rail for showers and bathtubs

Maximum load: max. 120 kg pull force

Classification: Grips, hand grips and holders for home use.

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